

# CLAVE<sup>®</sup> Universal Vial Spike

For Closed Preparation of Hazardous Drugs in any Size Vial



**Step 1 PREPARE** vial closure in accordance with manufacturer's instructions. Place vial on counter. Center spike of vial access device over stopper and push through stopper until locking arms grasp vial closure.



For 28mm vial closures, grasp locking arms with thumb and for finger to spread slightly while inserting spike into vial.




## Step 2

**INFUSE** diluent into drug vial if required. Vial must be in the upright position to infuse. Infusing fluid in the inverted position may damage filters.



SWAB CLAVE  
Before  
EVERY Access!

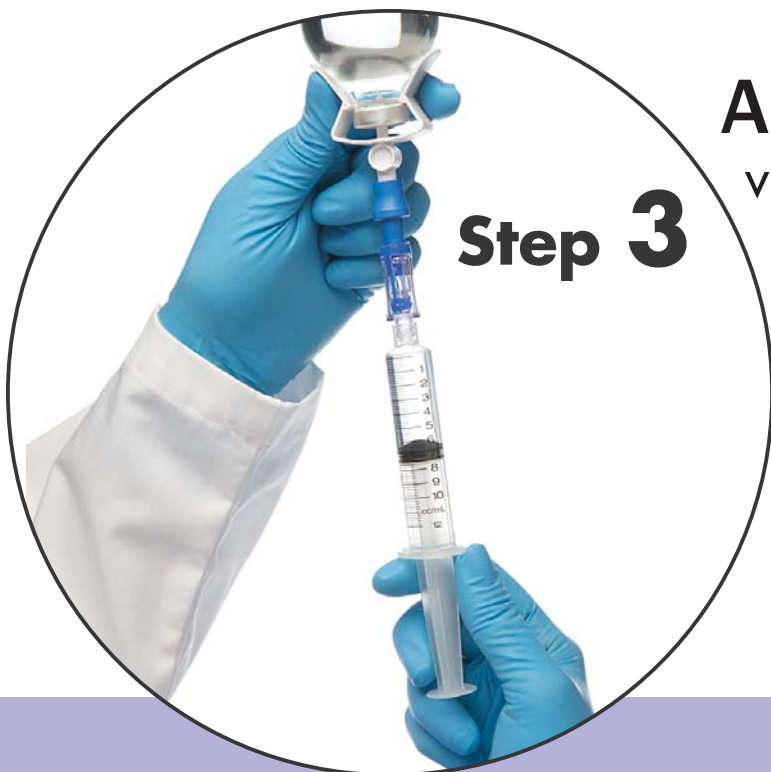

**ASPIRATE** drug from vial. Dual vents will facilitate drug flow and automatically equalize vial pressure. Universal vial spike may be used for multiple dosing. Swab CLAVE before every access.

