

MicroCLAVE[®]

Connector



PRIME MicroCLAVE by inverting the device to expel air.



SWAB MicroCLAVE before each use in accordance with facility protocol.




FLUSH MicroCLAVE after each use in accordance with facility protocol.



MicroCLAVE® Neutral Displacement Connector


For Needleless Access to Central, Peripheral, and Arterial Lines

- ➔ Flows more than 165 mL per minute on average at 36" head height.
- ➔ Holds 75 PSI backpressure.
- ➔ Ultra low residual volume of 0.04 mL (deadspace).
- ➔ Compatible with IV solutions, most chemotherapy drugs, and lipids.
- ➔ Contains no natural rubber latex.

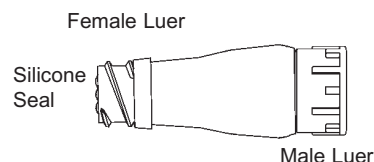


DIRECTIONS FOR USE

1. Using aseptic technique remove MicroCLAVE from package. Do not contaminate.
2. Prime MicroCLAVE in accordance with facility protocol. Invert device to expel air.
3. Remove protective cap and attach male luer of MicroCLAVE to extension set or IV catheter.

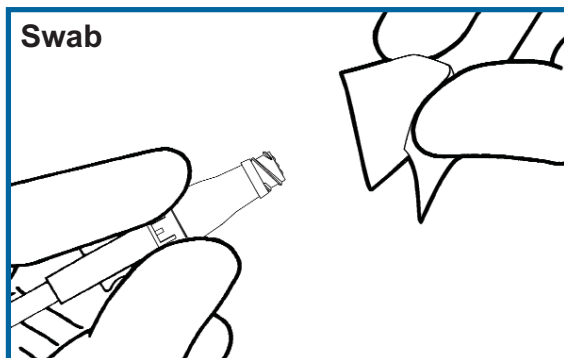


4. To access MicroCLAVE, swab silicone seal in accordance with facility protocol.
5. Attach a fully primed syringe or administration set to MicroCLAVE. Grasp MicroCLAVE and firmly push and twist male adapter into MicroCLAVE until secure.

NOTE: If using a rotating luer device, first push and twist luer slip into Micro CLAVE until secure, then lock down the rotating collar. This will ensure a secure connection and optimal flow rates.

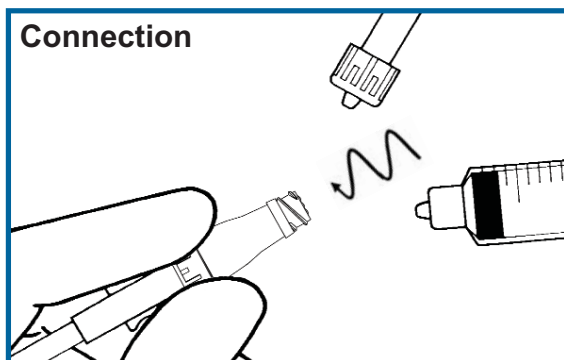


6. To disconnect, grasp MicroCLAVE and twist syringe or administration set away from MicroCLAVE until loose.

NOTE: Hold MicroCLAVE, not catheter hub to prevent accidental disconnection of MicroCLAVE from catheter.

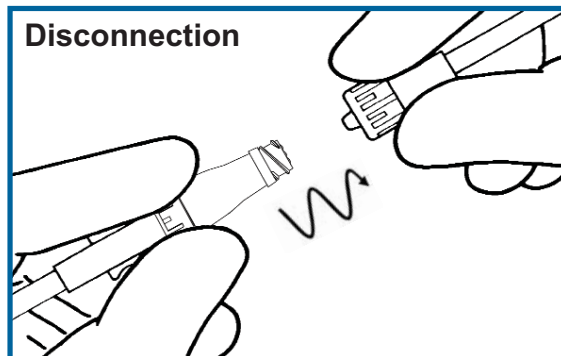

7. Flush MicroCLAVE with normal saline or in accordance with facility protocol after each use.
8. For subsequent connections repeat from step four (4).

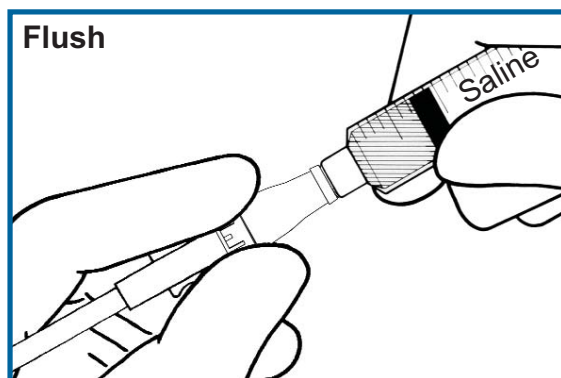

CAUTIONS:

- Do not use needles in MicroCLAVE.
- Do not cap MicroCLAVE, device is closed.